

QuickVue+® One-Step hCG Combo Quiz

Circle the correct answer

This quiz is an educational tool intended to assist facilities in evaluating their operators' understanding of the QuickVue+ One-Step hCG Combo test procedure. This quiz is not intended to be used as sole evidence of operator training or competency. Facilities are responsible for ensuring the quality of the testing performed by their operators. When testing controls or patient specimens, follow the current package insert instructions and/or procedure card included in the kit.

-
- 1. How often does the manufacturer recommend that external controls be run on the kit?**
- a) Once every 20 tests
 - b) Once per each new shipment of kits
 - c) Every first sample tested each day
 - d) Once for each shipment of a new kit lot number, and as otherwise required by your laboratory's quality assurance plan
-
- 2. At what temperature do the kits need to be stored?**
- a) 2-8°C (refrigerated)
 - b) 15-30°C (room temp.)
 - c) 2-30°C (refrigerated or room temp.)
 - d) -20°C (frozen)
-
- 3. Which external quality controls are recommended to be used with this kit?**
- a) Known patient samples
 - b) External quality controls are not recommended to be performed
 - c) Any manufacturer's external quality controls
 - d) hCG Control Set (Cat# 00272) or Serum hCG Control Set (Cat# 00281) – (each sold separately)
-
- 4. What is the read time for this kit?**
- a) 3 minutes for urine and serum
 - b) 5 minutes for urine and 3 minutes for serum
 - c) 10 minutes for serum and urine
 - d) 3 minutes for urine and 5 minutes for serum
-
- 5. Can the test be read after the read time?**
- a) No
 - b) Yes, up to 10 minutes after the read time
 - c) Yes, up to 5 minutes after the read time
 - d) Yes, any time after the read time
-
- 6. How long can the urine and serum specimens be stored refrigerated before being used to test?**
- a) The specimens must be used immediately
 - b) 8 hours for urine and 72 hours for serum
 - c) 72 hours for urine and 48 hours for serum
 - d) 24 hours for both urine and serum
-
- 7. What is the detection level of this test?**
- a) 50 mIU/ml for urine and 25 mIU/ml for serum
 - b) 20 mIU/ml for both urine and 10 mIU/ml for serum
 - c) 25 mIU/ml for urine and 10mIU/ml for serum
 - d) 10 mIU/ml for serum and 25 mIU/ml for urine
-
- 8. How many drops of serum or urine are needed to perform the test?**
- a) 5 drops
 - b) All of the serum or urine is needed
 - c) 4 drops
 - d) 3 drops
-
- 9. Can plasma be used with this test?**
- a) Yes, only if a sodium heparin (green top) tube is used
 - b) Yes, with any type of tubes
 - c) Yes, if only a potassium EDTA (lavender top) tube is used
 - d) No, plasma is not an approved specimen type
-
- 10. What is the "gold standard" test in detecting pregnancy?**
- a) Beta hCG Quantitative test
 - b) There is no "gold standard" test
 - c) Any rapid hCG test
 - d) Culture

QuickVue+® One-Step hCG Combo Quiz

Circle the correct answer

This quiz is an educational tool intended to assist facilities in evaluating their operators' understanding of the QuickVue+ One-Step hCG Combo test procedure. This quiz is not intended to be used as sole evidence of operator training or competency. Facilities are responsible for ensuring the quality of the testing performed by their operators. When testing controls or patient specimens, follow the current package insert instructions and/or procedure card included in the kit.

1. How often does the manufacturer recommend that external controls be run on the kit?

- a) Once every 20 tests
- b) Once per each new shipment of kits
- c) Every first sample tested each day
- d) Once for each shipment of a new kit lot number, and as otherwise required by your laboratory's quality assurance plan

2. At what temperature do the kits need to be stored?

- a) 2-8°C (refrigerated)
- b) 15-30°C (room temp.)
- c) 2-30°C (refrigerated or room temp.)
- d) -20°C (frozen)

3. Which external quality controls are recommended to be used with this kit?

- a) Known patient samples
- b) External quality controls are not recommended to be performed
- c) Any manufacturer's external quality controls
- d) hCG Control Set (Cat# 00272) or Serum hCG Control Set (Cat# 00281) – (each sold separately)

4. What is the read time for this kit?

- a) 3 minutes for urine and serum
- b) 5 minutes for urine and 3 minutes for serum
- c) 10 minutes for serum and urine
- d) 3 minutes for urine and 5 minutes for serum

5. Can the test be read after the read time?

- a) No
- b) Yes, up to 10 minutes after the read time
- c) Yes, up to 5 minutes after the read time
- d) Yes, any time after the read time

6. How long can the urine and serum specimens be stored refrigerated before being used to test?

- a) The specimens must be used immediately
- b) 8 hours for urine and 72 hours for serum
- c) 72 hours for urine and 48 hours for serum
- d) 24 hours for both urine and serum

7. What is the detection level of this test?

- a) 50 mIU/ml for urine and 25 mIU/ml for serum
- b) 20 mIU/ml for urine and 10 mIU/ml for serum
- c) 25 mIU/ml for urine and 10mIU/ml for serum
- d) 10 mIU/ml for serum and 25 mIU/ml for urine

8. How many drops of serum or urine are needed to perform the test?

- a) 5 drops
- b) All of the serum or urine is needed
- c) 4 drops
- d) 3 drops

9. Can plasma be used with this test?

- a) Yes, only if a sodium heparin (green top) tube is used
- b) Yes, with any type of tubes
- c) Yes, if only a potassium EDTA (lavender top) tube is used
- d) No, plasma is not an approved specimen type

10. What is the "gold standard" test in detecting pregnancy?

- a) Beta hCG Quantitative test
- b) There is no "gold standard" test
- c) Any rapid hCG test
- d) Culture