

QuickVue® iFOB Quiz

Circle the correct answer

This quiz is an educational tool intended to assist facilities in evaluating their operators' understanding of the QuickVue iFOB (immunochemical Fecal Occult Blood) test procedure. This quiz is not intended to be used as sole evidence of operator training or competency. Facilities are responsible for ensuring the quality of the testing performed by their operators. When testing controls or patient specimens, follow the current package insert instructions and/or procedure card included in the kit.

1. How often does the manufacturer recommend that external controls be run on the kit?

- a) Once every 20 tests
- b) Once per each new shipment of kits
- c) Every first sample tested each day
- d) Good laboratory practice recommends the use of external quality controls to assure the functionality of reagents and proper performance of the test procedure

2. At what temperature do the kits need to be stored?

- a) 2-8°C (refrigerated)
- b) 15-30°C (room temp.)
- c) 2-30°C (refrigerated or room temp.)
- d) -20°C (frozen)

3. How many times does a stool specimen need to be collected for this test?

- a) Only 1 time
- b) At least 2 times
- c) 3 times
- d) As many as needed

4. What is the read time for this kit?

- a) 5-10 minutes
- b) 10 minutes
- c) 5 minutes
- d) 15 minutes

5. Can the test be read after the read time?

- a) No
- b) Yes, up to 10 minutes after the read time
- c) Yes, up to 5 minutes after the read time
- d) Yes, any time after the read time

6. How long can the collected specimen be stored at ambient temperatures (below 95°F) before used to test?

- a) Up to 24 hours
- b) Up to 48 hours
- c) Up to 8 days
- d) Up to 6 months

7. Can a visibly bloody stool sample be collected to be used on this test?

- a) Yes, only if it's not too bloody
- b) Yes, if collected away from the bloody areas
- c) No, this test is intended for occult "hidden" blood
- d) Yes, if sample is diluted

8. How many drops of specimen solution are needed to perform the test?

- a) 5 drops
- b) 4 drops
- c) 6 drops
- d) 7 drops

9. How many times does the stool sample need to be pierced with the Sampler?

- a) 2 times
- b) 10 times
- c) As many as you want
- d) At least 5 times

10. Are there any dietary restrictions with this test?

- a) No
- b) Yes, the patient cannot eat red meat prior to collecting sample
- c) Yes, the patient cannot take Vitamin C supplements prior to collecting sample
- d) Yes, the patient cannot eat raw vegetables prior to collecting sample

QuickVue® iFOB Quiz

Circle the correct answer

This quiz is an educational tool intended to assist facilities in evaluating their operators' understanding of the QuickVue iFOB (immunochemical Fecal Occult Blood) test procedure. This quiz is not intended to be used as sole evidence of operator training or competency. Facilities are responsible for ensuring the quality of the testing performed by their operators. When testing controls or patient specimens, follow the current package insert instructions and/or procedure card included in the kit.

1. How often does the manufacturer recommend that external controls be run on the kit?

- a) Once every 20 tests
- b) Once per each new shipment of kits
- c) Every first sample tested each day
- d) Good laboratory practice recommends the use of external quality controls to assure the functionality of reagents and proper performance of the test procedure

2. At what temperature do the kits need to be stored?

- a) 2-8°C (refrigerated)
- b) 15-30°C (room temp.)
- c) 2-30°C (refrigerated or room temp.)
- d) -20°C (frozen)

3. How many times does a stool specimen need to be collected for this test?

- a) Only 1 time
- b) At least 2 times
- c) 3 times
- d) As many as needed

4. What is the read time for this kit?

- a) 5-10 minutes
- b) 10 minutes
- c) 5 minutes
- d) 15 minutes

5. Can the test be read after the read time?

- a) No
- b) Yes, up to 10 minutes after the read time
- c) Yes, up to 5 minutes after the read time
- d) Yes, any time after the read time

6. How long can the collected specimen be stored at ambient temperatures (below 95°F) before used to test?

- a) Up to 24 hours
- b) Up to 48 hours
- c) Up to 8 days
- d) Up to 6 months

7. Can a visibly bloody stool sample be collected to be used on this test?

- a) Yes, only if it's not too bloody
- b) Yes, if collected away from the bloody areas
- c) No, this test is intended for occult "hidden" blood
- d) Yes, if sample is diluted

8. How many drops of specimen solution are needed to perform the test?

- a) 5 drops
- b) 4 drops
- c) 6 drops
- d) 7 drops

9. How many times does the stool sample need to be pierced with the Sampler?

- a) 2 times
- b) 10 times
- c) As many as you want
- d) At least 5 times

10. Are there any dietary restrictions with this test?

- a) No
- b) Yes, the patient cannot eat red meat prior to collecting sample
- c) Yes, the patient cannot take Vitamin C supplements prior to collecting sample
- d) Yes, the patient cannot eat raw vegetables prior to collecting sample