

# QUICKVUE<sup>®</sup>

Influenza A+B TEST


## Test and Treat Today!

Diagnosing influenza can be difficult. One third of all upper respiratory infections are caused by influenza. This leads to approximately 36,000 deaths, over 200,000 hospitalizations<sup>1</sup> and accounts for \$10-\$15 billion spent in lost work and related medical expenses each year in the U.S.<sup>2</sup>

Quidel's CLIA-waived QuickVue<sup>®</sup> Influenza A+B test provides you with differential A or B test results allowing you to test and treat in just one visit. Prompt type-specific diagnosis enables a choice of antiviral therapy, helps avoid

the overuse of antibiotics, and prevents healthcare costs related to unnecessary testing and treatment.<sup>3</sup>

The QuickVue Influenza A+B test is CLIA waived, sensitive and specific, giving you the results you need. Right. Now.

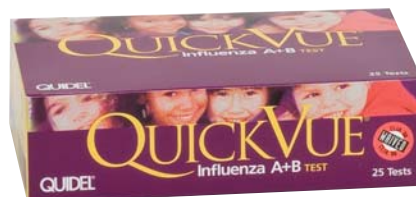

<sup>1</sup><http://www.cdc.gov/flu/keyfacts.htm>

<sup>2</sup>Rothberg, M.B. Management of Influenza Symptoms in Healthy Adults, J. Gen Intern Med 2003; 18:808-815.

<sup>3</sup>Montalto, N.J., D.O. An Office-Based Approach to Influenza: Clinical Diagnosis and Laboratory Testing, Am Fam Phys 2003; 67(1):111-118.

### Rapid Diagnosis and Treatment

- Identifies influenza Type A or B in positive samples, in about 10 minutes, for better patient management decisions.
- Most patients tested can be diagnosed and treated in just one visit.

### Accurate Detection

- QuickVue<sup>®</sup> Influenza A+B test detects both types A and B viral antigens.
- Accurate detection with nasal swab samples: 94% sensitivity and 90% specificity for type A 70% sensitivity and 97% specificity for type B
- See reverse side for additional performance claims.

### Easy to Use

- 3 steps  
1 reagent, 90 seconds hands-on time.
- 1 test procedure provides results for both A and B.

### Convenient Packaging

- Each kit contains everything needed to perform the test, including foam-tipped nasal swabs and Positive and Negative Controls.
- Available in 25-test kit.
- Room temperature storage with 24-month shelf life from date of manufacture.
- Transport swab set also available separately.


**QUIDEL**<sup>®</sup>  
CORPORATION

For more information on QuickVue<sup>®</sup> Influenza A+B test kits, visit our website or call your Quidel Account Manager at 1.800.874.1517 (U.S. only), or 858.552.1100 (outside U.S.).

Quidel Corporation • Worldwide Headquarters • 10165 McKellar Court • San Diego, CA 92121 USA • [www.quidel.com](http://www.quidel.com)



## Nasal/Nasopharyngeal swab procedure:

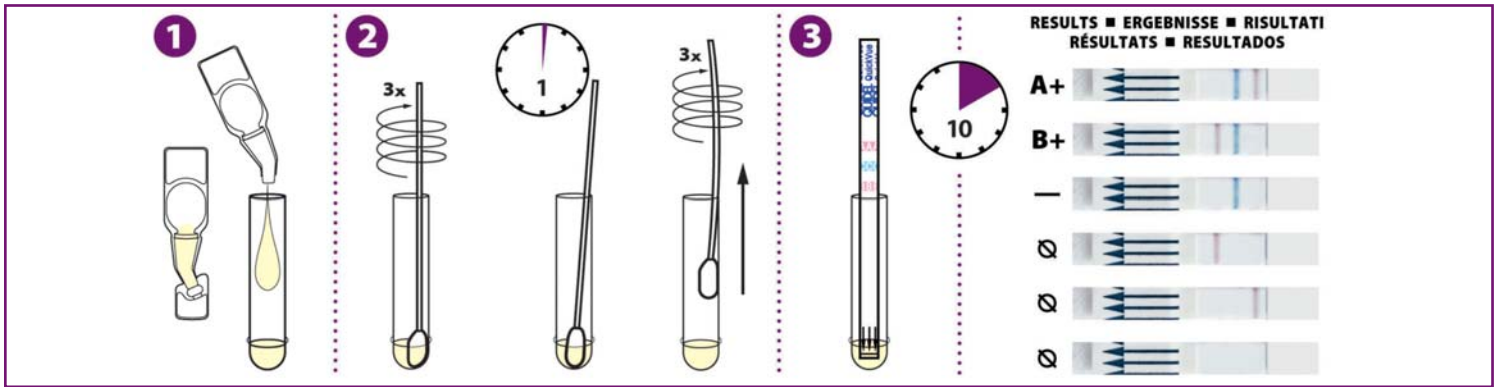

### Nasal Swab:

Sensitivity: A – 94%, B – 70%  
Specificity: A – 90%, B – 97%

Accuracy: A – 91%, B – 93%

PV (+): A – 62%, B – 82%

PV (-): A – 99%, B – 94%

### Nasopharyngeal Swab:


Sensitivity: A – 83%, B – 62%  
Specificity: A – 89%, B – 98%

Accuracy: A – 88%, B – 94%

PV (+): A – 67%, B – 80%

PV (-): A – 95%, B – 95%

## Nasal wash/aspirate procedure:

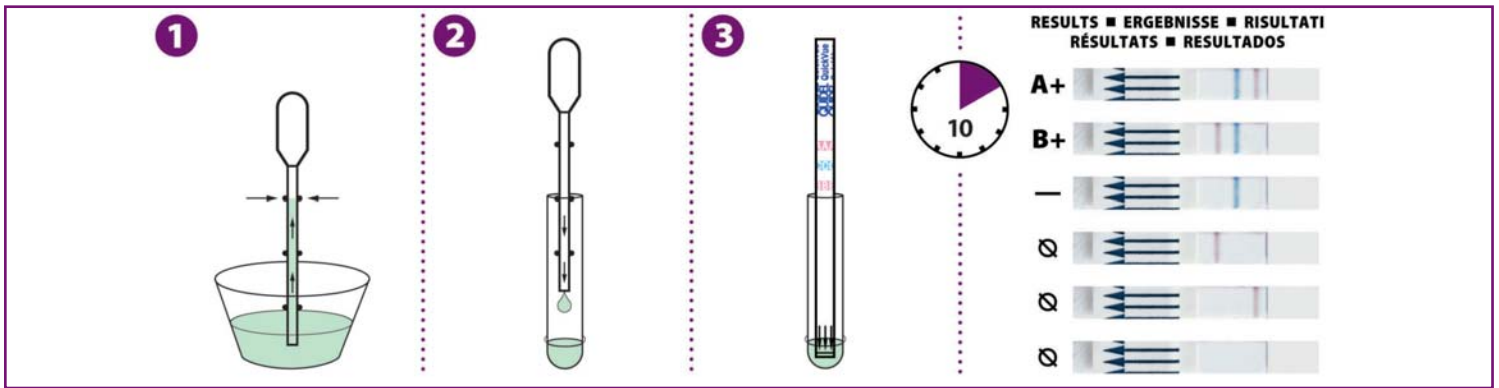

### Fresh nasal wash/aspirate:

Sensitivity: A – 77%, B – 82%  
Specificity: 99%

Accuracy: A – 95%, B – 96%

PV (+): A – 91%, B – 90%

PV (-): A – 96%, B – 97%

### Frozen nasal wash:

Sensitivity: A – 86%  
Specificity: A – 95%

Accuracy: A – 91%

PV (+): A – 93%

PV (-): A – 90%

The performance of any rapid flu test is dependent on sample collection and handling and the adherence to the Package Insert. Refer to our website at [www.quidel.com](http://www.quidel.com) for additional performance claims.



To order: Call your Quidel Account Manager or your local distributor

QuickVue® Influenza A+B – 25 Test Kit:  
Catalog #20183



QuickVue® Swab Transport Tube Set  
Catalog #20171



For more information on QuickVue® Influenza A+B test kits, visit our website or call your Quidel Account Manager at 1.800.874.1517 (U.S. only), or 858.552.1100 (outside U.S.).

Quidel Corporation • Worldwide Headquarters • 10165 McKellar Court • San Diego, CA 92121 USA • [www.quidel.com](http://www.quidel.com)

Distributor Information: