



### Netgear Ethernet Switch Package Contents

The following system components are supplied with the Netgear Ethernet Switch:

- Netgear Ethernet Switch
- Ethernet Cable(s)
- Power Supply
- Installation Instructions



\*Only authorized users approved by the Healthcare Provider's Organizational Administrator would have access to registering on MyVirena.com.

**Note:** If any item is missing or damaged, please contact Quidel Technical Support at 800.874.1517 (in the U.S.), 858.552.1100 (outside the U.S.) or e-mail [technicalsupport@quidel.com](mailto:technicalsupport@quidel.com).

### Netgear Ethernet Switch Installation Instructions

There are a few simple steps required in order to set up your Netgear Ethernet Switch with your Virena router:

1. **Connect the Ethernet Cable:** Connect the Ethernet cable from port 5 on the Netgear Ethernet Switch (Figure 1) to the Ethernet port on the Virena router (LAN 0 on Virena WR21).

Figure 1



2. **Connect the Quidel Instrument:** For each Quidel instrument to be connected to the Netgear Ethernet Switch, connect the Ethernet cable from port 1, 2, 3, or 4 on the Netgear Ethernet Switch to the Ethernet port on the Quidel instrument.
3. **Set Network Configuration:** Set the Network Configuration on the Quidel instrument as follows:

Port Number	1	2	3	4
Network IP Address	192.168.0.5	192.168.0.6	192.168.0.7	192.168.0.8

4. **Connect to Power Supply:** Connect the plug end of the 7.5 volt power supply to the power connector on the Netgear Ethernet Switch (Figure 2). Insert the power supply end fully. Then plug the other end into a wall outlet.
5. **Confirm the Power Switch is On:** Check the power switch on the Netgear Ethernet Switch (Figure 2) and confirm it is in the On position.

Figure 2

