



**A replacement component for Quidel Product 01-010000.v2 D<sup>3</sup> Ultra™ DFA Respiratory Virus Screening & ID Kit.**

## FOR *IN VITRO* DIAGNOSTIC USE



### INTENDED USE

Parainfluenza 3 DFA Reagent is a replacement component for Quidel Product 01-010000.v2 D<sup>3</sup> Ultra DFA Respiratory Virus Screening & ID Kit.

### MATERIALS PROVIDED

**Quidel Product 01-013702.v2 contains:**

#### **Parainfluenza 3 DFA Reagent** **2 mL**

One dropper bottle containing 2 mL of fluorescein labeled murine monoclonal antibodies directed against antigens produced by parainfluenza virus type 3. The buffered, stabilized, aqueous solution contains Evans Blue as a counter-stain and 0.1% sodium azide as preservative.

### STORAGE AND HANDLING

Reagent will retain its full potency through the expiration date when stored at 2°C to 8°C. Store in the dark.

### WARNINGS AND PRECAUTIONS

- For *in vitro* diagnostic use.
- Use undiluted.
- Refer to 01-010000.v2 Package Insert for complete directions.
- Sodium azide is included in this reagent at 0.1%.
  - ▶ Aqueous solutions of sodium azide, when mixed with acids, may liberate toxic gas.
  - ▶ Sodium azide may react with lead and copper plumbing to form highly explosive metal azides.
  - ▶ Avoid disposal of these solutions down sanitary or industrial plumbing systems.
  - ▶ Avoid release to the environment.
- Evans Blue is a potential carcinogen. If skin contact occurs, flush with water immediately.
- Testing should be performed in an area with adequate ventilation.
- Dispose of containers and unused contents in accordance with Federal, State and Local regulatory requirements.
- Wear suitable protective clothing, gloves, and eye/face protection when handling the contents of this kit.
- Wash hands thoroughly after handling.
- For additional information on hazard symbols, safety, handling and disposal of the components within this kit, please refer to the Safety Data Sheet (SDS) located at [quidel.com](http://quidel.com).

## ASSISTANCE

To place an order or for technical support, please contact a Quidel Representative at 800.874.1517 (in the U.S.) or 858.552.1100 (outside the U.S.), Monday through Friday, from 8:00 a.m. to 5:00 p.m., Eastern Time. Orders may also be placed by fax at 740.592.9820. For e-mail support contact [customerservice@quidel.com](mailto:customerservice@quidel.com) or [technicalsupport@quidel.com](mailto:technicalsupport@quidel.com).

For services outside the U.S.A., please contact your local distributor. Additional information about Quidel, our products, and our distributors can be found on our website [quidel.com](http://quidel.com).

REF

01-013702.v2 – Parainfluenza 3 DFA Reagent

IVD



EC REP

MDSS GmbH  
Schiffgraben 41  
30175 Hannover,  
Germany



**Diagnostic Hybrids, Inc. – a subsidiary of Quidel Corporation**  
2005 East State Street, Suite 100  
Athens, OH 45701 USA  
[quidel.com](http://quidel.com)

PI167P300EN00 (10/18)

## GLOSSARY

---

**REF**

Catalogue number



CE mark of conformity

---

**EC REP**

Authorized Representative  
in the European Community

**LOT**

Batch code

---



Use by



Manufacturer

---



Temperature limitation



Intended use

---



Consult e-labeling  
instructions for use



Do not reuse

---

**IVD**

For *In Vitro* diagnostic use

---