



**A replacement component for Quidel Product 01-050000 D<sup>3</sup> IFA Enterovirus Identification Kit.**

## FOR *IN VITRO* DIAGNOSTIC USE



### INTENDED USE

D<sup>3</sup> IFA Enterovirus Reagent Duo Pack is a replacement component for Quidel Product 01-050000 D<sup>3</sup> IFA Enterovirus Identification Kit.

### MATERIALS PROVIDED

**Quidel Product 01-050001 contains:**

**D<sup>3</sup> Enterovirus MAb Reagent**

**5 mL**

One bottle containing 5 mL of murine monoclonal antibodies directed against antigens produced by Enterovirus strains. The buffered, stabilized, aqueous solution contains 0.1% sodium azide as preservative.

**D<sup>3</sup> Anti-mouse Conjugate**

**5 mL**

One dropper bottle containing 5 mL of FITC labeled goat anti-mouse IgG. The buffered, stabilized, aqueous solution contains Evans Blue and 0.1% sodium azide as preservative.

### STORAGE AND HANDLING

Reagent will retain its full potency through the expiration date shown on the label when stored at 2°C to 8°C. Light exposure of the reagent should be kept to a minimum.

### WARNINGS/PRECAUTIONS

- Sodium azide is included in the solutions within this kit at 0.1%.
  - Aqueous solutions of sodium azide, when mixed with acids, may liberate toxic gas.
  - Sodium azide may react with lead and copper plumbing to form highly explosive metal azides. Avoid disposal of these solutions down sanitary or industrial plumbing systems.
  - Avoid release to the environment.
- Evans Blue is a potential carcinogen. If skin contact occurs, flush with water immediately.
- Reagents are to be distributed as a set and not individually.
- Testing should be performed in an area with adequate ventilation.
- Dispose of containers and unused contents in accordance with Federal, State and Local regulatory requirements.
- Wear suitable protective clothing, gloves, and eye/face protection when handling the contents of this kit.
- Wash hands thoroughly after handling.
- For additional information on hazard symbols, safety, handling and disposal of the components within this kit, please refer to the Safety Data Sheet (SDS) located at [quidel.com](http://quidel.com).

## INSTRUCTIONS

- Use undiluted.
- Refer to 01-050000 product insert for complete directions.

## WARRANTY STATEMENT

These products are warranted to perform as described in their labeling and the Quidel literature when used in accordance with their instructions. THERE ARE NO WARRANTIES WHICH EXTEND BEYOND THIS EXPRESS WARRANTY AND QUIDEL DISCLAIMS ANY IMPLIED WARRANTY OF MERCHANTABILITY OR WARRANTY OF FITNESS FOR PARTICULAR PURPOSE. Quidel sole obligation and purchaser's exclusive remedy for breach of this warranty shall be at the option of Quidel to repair or replace the products.

## ASSISTANCE

To place an order or for technical support, please contact a Quidel Representative at 800.874.1517 (in the U.S.) or 858.552.1100 (outside the U.S.), Monday through Friday, from 8:00 a.m. to 5:00 p.m., Eastern Time. Orders may also be placed by fax at (740) 592-9820. For e-mail support contact [customerservice@quidel.com](mailto:customerservice@quidel.com) or [technicalsupport@quidel.com](mailto:technicalsupport@quidel.com).

For services outside the U.S.A., please contact your local distributor. Additional information about Quidel, our products, and our distributors can be found on our website [quidel.com](http://quidel.com).

**REF** 01-050001 – D<sup>3</sup> IFA Enterovirus Reagent Duo

**IVD**



**EC REP**

MDSS GmbH  
Schiffgraben 41  
30175 Hannover,  
Germany



**Diagnostic Hybrids, Inc. – a subsidiary of Quidel Corporation**  
2005 East State Street, Suite 100  
Athens, OH 45701 USA  
[quidel.com](http://quidel.com)

**PI1781000EN00(06/18)**

---

**REF**

Catalogue number



CE mark of conformity

---

**EC REP**

Authorized Representative  
in the European Community

**LOT**

Batch code

---



Use by



Manufacturer

---



Temperature limitation



Intended use

---



Consult e-labeling  
instructions for use



Do not reuse

---

**IVD**

For *In Vitro* diagnostic use

**CONT NaN<sub>3</sub>**

Contents/Contains

---