



A replacement component for Quidel Product 01-050000 D³ IFA Enterovirus Identification Kit.

For in vitro diagnostic use.

A symbols glossary can be found at quidel.com/glossary.



INTENDED USE

D³ IFA Enterovirus Reagent Duo Pack is a replacement component for Quidel Product 01-050000 D³ IFA Enterovirus Identification Kit.

MATERIALS PROVIDED

Quidel Product 01-050001 contains:

D³ Enterovirus MAb Reagent

5 mL

One bottle containing 5 mL of murine monoclonal antibodies directed against antigens produced by Enterovirus strains. The buffered, stabilized, aqueous solution contains 0.1% sodium azide as preservative.

D³ Anti-mouse Conjugate

5 mL

One dropper bottle containing 5 mL of FITC labeled goat anti-mouse IgG. The buffered, stabilized, aqueous solution contains Evans Blue and 0.1% sodium azide as preservative.

STORAGE AND HANDLING

Reagent will retain its full potency through the expiration date shown on the label when stored at 2°C to 8°C. Light exposure of the reagent should be kept to a minimum.

WARNINGS/PRECAUTIONS

- Sodium azide is included in the solutions within this kit at 0.1%.
 - ▶ Aqueous solutions of sodium azide, when mixed with acids, may liberate toxic gas.
 - ▶ Sodium azide may react with lead and copper plumbing to form highly explosive metal azides. Avoid disposal of these solutions down sanitary or industrial plumbing systems.
 - ▶ Avoid release to the environment.
- Evans Blue is a potential carcinogen. If skin contact occurs, flush with water immediately.
- Reagents are to be distributed as a set and not individually.
- Testing should be performed in an area with adequate ventilation.
- Dispose of containers and unused contents in accordance with Federal, State and Local regulatory requirements.
- Wear suitable protective clothing, gloves, and eye/face protection when handling the contents of this kit.
- Wash hands thoroughly after handling.

- For additional information on hazard symbols, safety, handling and disposal of the components within this kit, please refer to the Safety Data Sheet (SDS) located at quidel.com.

INSTRUCTIONS

- Use undiluted.
- Refer to 01-050000 product insert for complete directions.

WARRANTY STATEMENT

These products are warranted to perform as described in their labeling and the Quidel literature when used in accordance with their instructions. THERE ARE NO WARRANTIES WHICH EXTEND BEYOND THIS EXPRESS WARRANTY AND QUIDEL DISCLAIMS ANY IMPLIED WARRANTY OF MERCHANTABILITY OR WARRANTY OF FITNESS FOR PARTICULAR PURPOSE. Quidel's sole obligation and purchaser's exclusive remedy for breach of this warranty shall be, at the option of Quidel, to repair or replace the products.

ASSISTANCE

To place an order or for technical support, please contact a Quidel Representative at 800.874.1517 (in the U.S.) or 858.552.1100 (outside the U.S.), Monday through Friday, from 8:00 a.m. to 5:00 p.m., Eastern Time. Orders may also be placed by fax at (740) 592-9820. For e-mail support contact customerservice@quidel.com or technicalsupport@quidel.com.

For services outside the U.S.A., please contact your local distributor. Additional information about Quidel, our products, and our distributors can be found on our website quidel.com.

REF 01-050001 – D³ IFA Enterovirus Reagent Duo

IVD



EC REP

MDSS GmbH
Schiffgraben 41
30175 Hannover,
Germany



Diagnostic Hybrids, Inc. – a subsidiary of Quidel Corporation
2005 East State Street, Suite 100
Athens, OH 45701 USA
quidel.com

PI1781001EN00 (03/21)

REF

Catalogue number



CE mark of conformity

EC REP

Authorized Representative
in the European Community

LOT

Batch code



Use by



Manufacturer



Temperature limitation



Intended use



Consult e-labeling
instructions for use



Do not reuse

IVD

For *In Vitro* diagnostic use

CONT NaN₃

Contents/Contains
