



QUIDEL

Antibiotic Solution

Additive when preparing cell culture media.

FOR *IN VITRO* DIAGNOSTIC USE

DESCRIPTION

One tube containing 3-mL of a combined antibiotic PBS solution at the following concentrations: 2 mg/mL Streptomycin Sulfate, 500 µg/mL Amphotericin B, 1 mg/mL Vancomycin, 250 units/mL Nystatin and 1 mg/mL Gentamicin; for use as an additive when preparing cell culture media. The solution has been tested for its compatibility with cell cultures.

WARNINGS AND PRECAUTIONS

- General purpose laboratory reagent.
- May be an eye, skin or respiratory irritant. For eye exposure, rinse with water for several minutes. For skin exposure, wash with soap and water. For respiratory difficulties, remove to fresh air. Seek medical attention if any condition persists.
- Testing should be performed in an area with adequate ventilation.
- Dispose of containers and unused contents in accordance with Federal, State and Local regulatory requirements.
- Wear suitable protective clothing, gloves, and eye/face protection when handling the contents of this kit.
- Wash hands thoroughly after handling.
- For additional information on hazard symbols, safety, handling and disposal of the components within this kit, please refer to the Safety Data Sheet (SDS) located at quidel.com.

STORAGE

Reagent will retain its full potency through the expiration date when stored at 2° to 8°C. Store in the dark.

INSTRUCTIONS

Dilute to appropriate concentration. Recommended dilution scheme is to dilute product 1:10 in Universal Transport Medium.

ASSISTANCE

To place an order or for technical support, please contact a Quidel Representative at 800.874.1517 (in the U.S.) or 858.552.1100 (outside the U.S.), Monday through Friday, from 8:00 a.m. to 5:00 p.m., Eastern Time. Orders may also be placed by fax at (740) 592-9820. For e-mail support contact customerservice@quidel.com or technicalsupport@quidel.com.

For services outside the U.S.A., please contact your local distributor. Additional information about Quidel, our products, and our distributors can be found on our website quidel.com.

REF

10-040003 – Antibiotic Solution

IVD



Diagnostic Hybrids, Inc. – a subsidiary of Quidel Corporation

2005 East State Street, Suite 100

Athens, OH 45701 USA

quidel.com

PI9080000EN00 (04/18)

GLOSSARY



Catalogue number



Batch code



Use by



Manufacturer



Temperature limitation



Intended use



Consult e-labeling instructions for use



Do not reuse



Biological risks



For *In Vitro* diagnostic use



Contents/Contains
