

Circle the correct answer

This quiz is an educational tool intended to assist facilities in evaluating their operators' understanding of the InflammaDry procedure. This quiz is not intended to be used as sole evidence of operator training or competency. Facilities are responsible for ensuring the quality of the testing performed by their operators. When testing controls or patient specimens, follow the current Package Insert instructions.

1. How often does the manufacturer recommend that InflammaDry external controls be run?

- a) Once with each new lot number
- b) Once with each new shipment received
- c) Every 30 days
- d) All of the above

2. What shows in the result window of an unused test?

- a) Two faint orange lines
- b) One blue control line
- c) One blue control line and one red result line
- d) The result window is completely white (no lines)

3. At what temperature do the kits need to be stored?

- a) 2°C to 8°C
- b) 4°C to 25°C
- c) 2°C to 30°C
- d) 0°C

4. MMP-9 may be elevated in the tears of patients with dry eye.

- a) True
- b) False

5. When collecting a sample from the patient you should:

- a) Dab the sample collector in multiple locations 6 to 8 times
- b) Dab the sample collector once
- c) Rest the sample collector against the conjunctiva for 5 seconds after dabbing
- d) Both A and C

6. If ocular anesthetic or any other topical medication is applied to the eye, how long should you wait before collecting the sample?

- a) At least 5 minutes
- b) At least 20 minutes
- c) At least 1 hour
- d) At least 2 hours

7. If the fleece is not saturated and glistening after collecting a sample, you should:

- a) Run the test
- b) Continue to dab the sampling fleece along the palpebral conjunctiva until fleece glistens
- c) Throw the test away
- d) None of the above

8. Which of the following is true with regard to reading the results?

- a) The test should be read within 1 hour
- b) The test should be read within 3 hours
- c) The test should be read within 6 hours
- d) The test should be read within 9 hours

9. The InflammaDry test and buffer vial are multiple use items.

- a) True
- b) False

10. The sample collector is a sterile component.

- a) True
- b) False

11. What shows in the result window when the test result is positive?

- a) Two faint orange lines
- b) One blue control line
- c) One blue control line and one red result line
- d) The result window is completely white (no lines)

12. What shows in the result window when the test result is negative?

- a) Two faint orange lines
- b) One blue control line
- c) One blue control line and one red result line
- d) The result window is completely white (no lines)

Circle the correct answer

This quiz is an educational tool intended to assist facilities in evaluating their operators' understanding of the InflammaDry procedure. This quiz is not intended to be used as sole evidence of operator training or competency. Facilities are responsible for ensuring the quality of the testing performed by their operators. When testing controls or patient specimens, follow the current Package Insert instructions.

-
- 1. How often does the manufacturer recommend that InflammaDry external controls be run?**
- a) Once with each new lot number c) Every 30 days
b) Once with each new shipment received d) All of the above
-
- 2. What shows in the result window of an unused test?**
- a) Two faint orange lines c) One blue control line and one red result line
b) One blue control line d) The result window is completely white (no lines)
-
- 3. At what temperature do the kits need to be stored?**
- a) 2°C to 8°C c) 2°C to 30°C
 b) 4°C to 25°C d) 0°C
-
- 4. MMP-9 may be elevated in the tears of patients with dry eye.**
- a) True b) False
-
- 5. When collecting a sample from the patient you should:**
- a) Dab the sample collector in multiple locations 6 to 8 times c) Rest the sample collector against the conjunctiva for 5 seconds after dabbing
b) Dab the sample collector once d) Both A and C
-
- 6. If ocular anesthetic or any other topical medication is applied to the eye, how long should you wait before collecting the sample?**
- a) At least 5 minutes c) At least 1 hour
b) At least 20 minutes d) At least 2 hours
-
- 7. If the fleece is not saturated and glistening after collecting a sample, you should:**
- a) Run the test c) Throw the test away
 b) Continue to dab the sampling fleece along the palpebral conjunctiva until fleece glistens d) None of the above
-
- 8. Which of the following is true with regard to reading the results?**
- a) The test should be read within 1 hour c) The test should be read within 6 hours
b) The test should be read within 3 hours d) The test should be read within 9 hours
-
- 9. The InflammaDry test and buffer vial are multiple use items.**
- a) True b) False
-
- 10. The sample collector is a sterile component.**
- a) True b) False
-
- 11. What shows in the result window when the test result is positive?**
- a) Two faint orange lines c) One blue control line and one red result line
b) One blue control line d) The result window is completely white (no lines)
-
- 12. What shows in the result window when the test result is negative?**
- a) Two faint orange lines c) One blue control line and one red result line
b) One blue control line d) The result window is completely white (no lines)