

Objective Strep A Results at Your Fingertips



Accurate, objective and automated results in **5 minutes**



Secure, remote instrument management



Excellent performance compared to molecular and bacterial culture methods



Integrated data management automatically stores test and user history



Flexible, dual mode testing for a variety of laboratory environments




User and patient ID captured with **onboard barcode scanner**



Sofia Strep A+ FIA – Procedures



Throat Swab Procedure

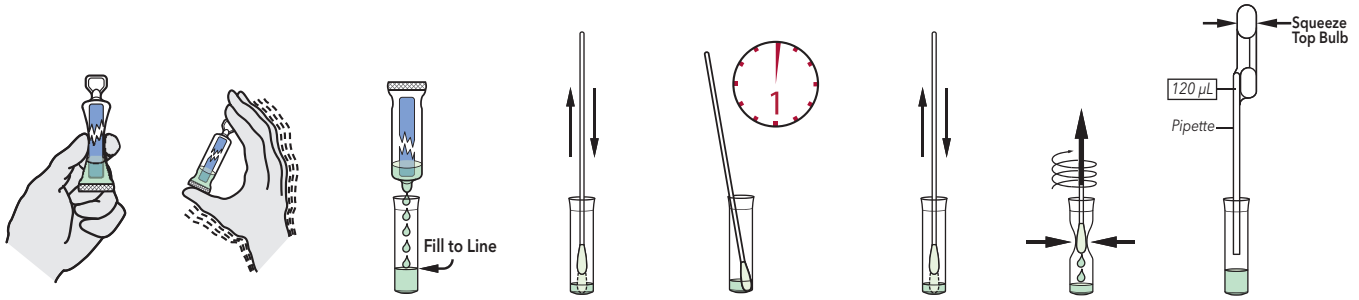

Sofia 2 Development Modes

WALK AWAY Mode – Walk away and multitask

READ NOW Mode – Batch multiple samples per hour

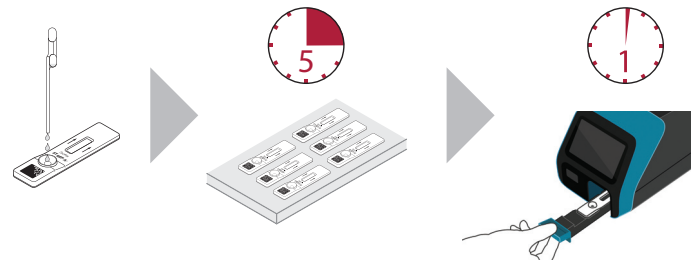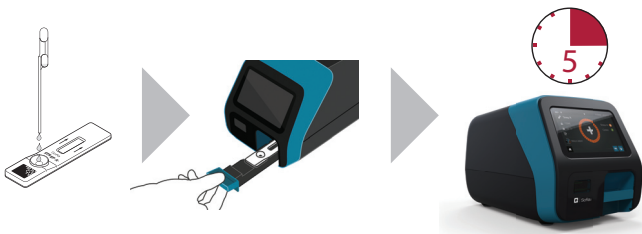

Sofia Strep A+ FIA

- Accurate detection with throat swab samples
- Excellent performance demonstrated in a multi-site study*
- Compatible with viral transport media
- Room temperature storage
- 24-month shelf life from date of manufacture

Sofia Strep A+ FIA Results vs. Culture*

	Culture			
	Pos	Neg		
Sofia Pos	164	38**	Sensitivity	93.7% (164/175)
Sofia Neg	11***	638	Specificity	94.4% (638/676)
Total	175	676		

**Of the 38 discordant specimens, 24 of these specimens were positive for GAS when tested with an FDA-cleared molecular device, 14 were negative.

***Of the 11 discordant specimens, 3 were negative when tested with an FDA-cleared molecular device, 8 were positive.

*Refer to the Package Insert for additional performance claims.

