



Sofia<sup>®</sup>  
SARS Antigen FIA



# Objective SARS-CoV-2 Results at Your Fingertips



Accurate, objective and automated results in **15 minutes**



**Integrated data management** automatically stores test and user history



**Excellent performance** compared to molecular methods



User and patient ID captured with **onboard barcode scanner**




**Flexible, dual mode testing** for high throughput in variety of laboratory environments



# Sofia SARS Antigen FIA – Procedure


## Nasal/Nasopharyngeal Swab Procedure

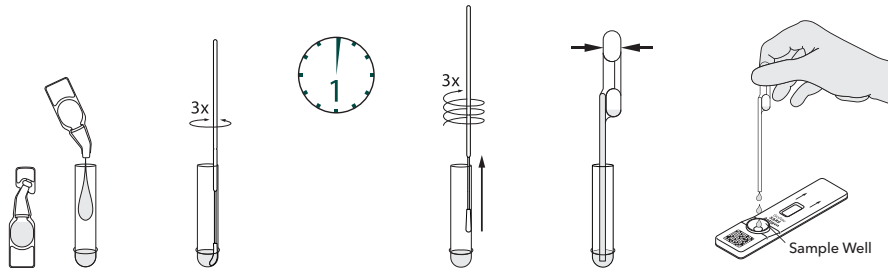

## Sofia Development Modes

**WALK AWAY Mode – Walk away and multitask**

**READ NOW Mode – Batch multiple samples per hour**

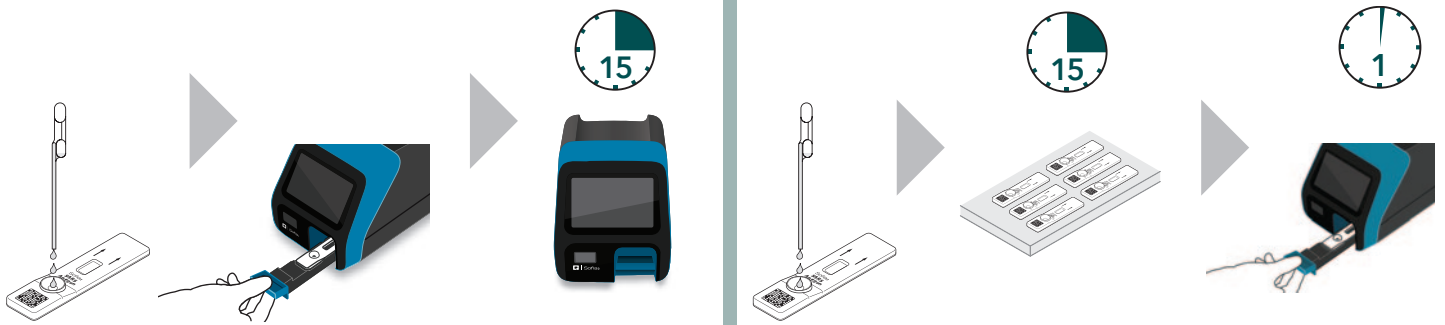

## Sofia SARS Antigen FIA

- Nasal and nasopharyngeal specimens
- Accurate detection with direct samples
- Results in 15 minutes
- Kit includes 25 sterile nasal swabs and a set of positive and negative control swabs
- Room temperature storage

	Reference Extracted RT-PCR assay					95% CI		
	POS	NEG	Total	PPA	96.7%	83.3%	99.4%	
<b>Sofia SARS Antigen FIA Assay</b>	POS	29	0	29	NPA	100.0%	97.9%	100.0%
	NEG	1	179	180	PPV	100.0%	88.3%	100.0%
	Total	30	179	209	NPV	99.4%	96.9%	99.9%
				Prevalence	14.4%	7.4%	14.3%	
				% agreement	99.5%			

\*Refer to the Package Insert.



SL3206 | SS20378101EN00 (08/20)

Sofia SARS Antigen FIA – 25 Test Kit: Catalog #20378

