



QuickVue®

Technical Bulletin

QuickVue Influenza A+B Test and QuickVue RSV Test: Test for Both Influenza and RSV Using One Patient Sample

Note: This procedure is not intended for use with QuickVue RSV 10 (Cat. #20222) Test kit.




Study the Package Inserts thoroughly for both the QuickVue Influenza A+B Test and QuickVue RSV Test. The following is not a complete set of instructions.

Nasopharyngeal Swab Specimens

Using one nasopharyngeal swab (NPS) specimen, elute the NPS in 650-1000 μL of saline or indicated media by vigorously rotating the swab in the diluent. Then keep swab in the diluent for 1 minute. Follow the sample preparation procedure shown below and the test procedures in the QuickVue Influenza A+B and QuickVue RSV Package Inserts.

Test Procedure for Nasopharyngeal Swab Specimens

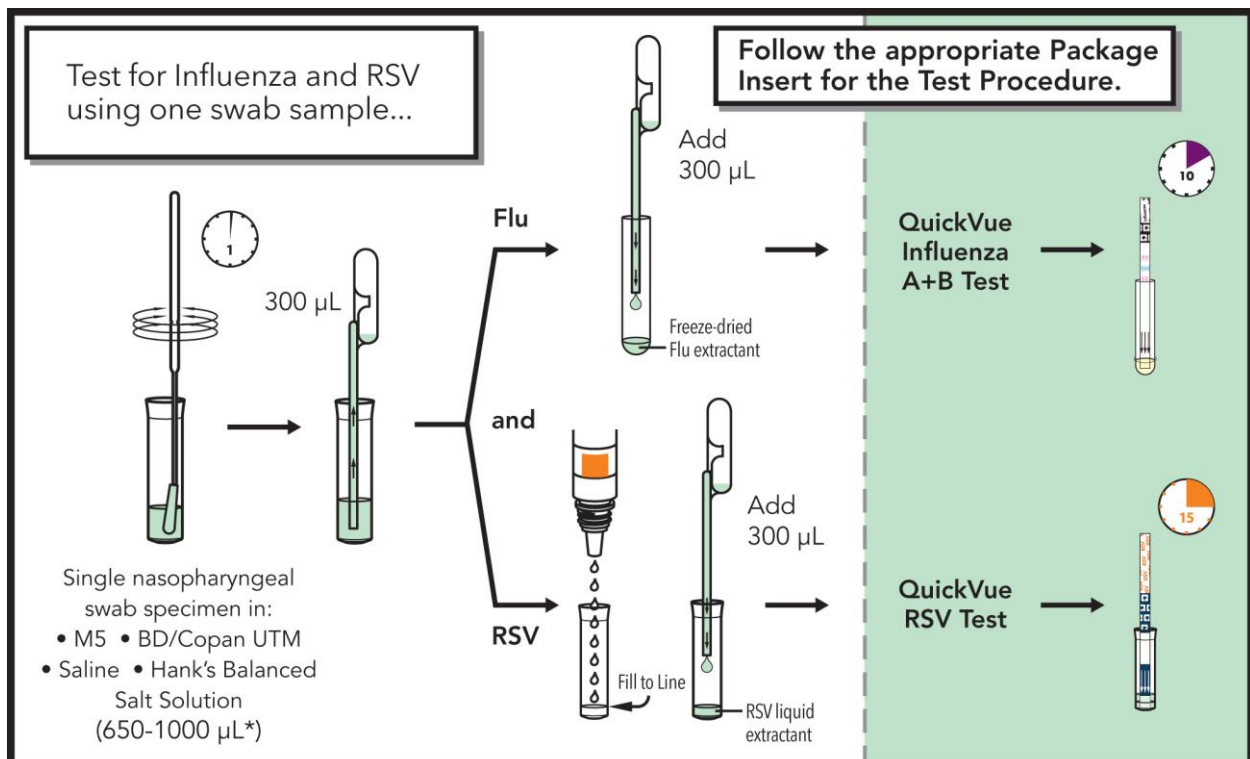

*Use minimal amount of saline or media, as excess volume will dilute the amount of antigen in the specimen. If transport to the lab is required for cell culture, ensure adequate volume of the specimen is available.

NOTE: QuickVue Influenza A+B Test performance testing Swab samples diluted in transport media has not been evaluated and established in clinical studies.

If necessary, please contact Quidel Technical Support for specimen tubes or saline vials.

If you have any questions regarding the use of this product, please contact Quidel Technical Support at 1.800.874.1517 (in the U.S.) or technicalsupport@quidel.com. If outside the U.S., further information can be obtained from your distributor, or directly from Quidel at one of the numbers listed below. Reference quidel.com to see more options for Support.

Country	Phone	E-Mail Address
Europe, Middle East and Africa	+353 (91) 412 474 (main) 0 1800 200441 (toll free)	emeatechnicalsupport@quidel.com
Austria	+43 316 231239	
France	0 (805) 371674	
Germany	+49 (0) 7154 1593912	
Netherlands	0 800 0224198	
Switzerland	0 800 554864	
United Kingdom	0 800 3688248	
Italy	+39 (800) 620 549	
North America, Asia-Pacific, Latin America	858.552.1100	technicalsupport@quidel.com
Canada	437.266.1704 (main) 888.415.8764 (toll free)	technicalsupport@quidel.com
China	0400 920 9366 or +86 021 3217 8300	chinatechnicalservice@quidel.com

You may also visit our website at quidel.com for information on Quidel's line of Rapid Diagnostics, Molecular Diagnostics, Cell Culture and Specialty Products (Bone Health and Autoimmune & Complement). Other product information available on our website includes: CPT codes, CLSI procedure guides, SDS, and Package Inserts.

TB1008300EN00 (10/19)