



Sofia®

Technical Bulletin

Sofia Printer Roller Installation

In the event that you need to replace the Printer Roller, please see the figure below for proper installation. It is important to note that the roller gear/cog be located on the left hand side of the cover (when snapped in place) as displayed in the picture below. This will ensure that it can mesh with the corresponding gear for printer function.



Please contact Quidel Technical Support at 800.874.1517 (in the U.S.), 858.552.1100 (outside the U.S.) or technicalsupport@quidel.com if you have any questions regarding Sofia, or any Quidel product. Our hours of operation are Monday through Friday, 7:00 a.m. to 5:00 p.m. Pacific Time.

You may also visit our website at quidel.com for information on Quidel's line of Rapid Diagnostics, Molecular Diagnostics, Cell Culture and Specialty Diagnostics (Bone Health and Autoimmune & Complement). Other product information available on our website includes: CPT codes, CLSI procedure guides, MSDS, and Package Inserts.